BioStrip LH Ovulation Test

Catalogue # 4220-13

INTENDED USE

The BioStrip LH ovulation test is designed for the in vitro detection of human luteinizing hormone (LH) in urine to predict the time of ovulation (the release of mature ovum from the ovary). For professional use only.

PRINCIPLE AND EXPLANATION OF PROCEDURE

Pregnancy is the result of fertilization of mature ovum with sperm. Ovulation only occurs once during a menstrual cycle. Timing of intercourse during ovulation may increase the chance of pregnancy.

Human luteinizing hormone (LH) normally present in small amount in urine plays important roles in the regulating the ovulation and ovarian functions during the menstrual cycle. About 12-30 hours before a mature ovum is released from the ovarian follicle, the level of LH rises rapidly (LH surge) and peaks during mid-cycle.

The BioStrip LH Ovulation Test detects this LH surge telling you that ovulation is likely to occur in the next 24-48 hours.

WARNING AND PRECAUTIONS

1. Read this entire pamphlet carefully and make sure you are familiar with the contents of this test kit.
2. Do not use test after expiry date stamped on the package.
3. Do not open foil pouch until ready to perform test.
4. Do not re-use test unit.
5. For single use only, do not reuse.

STORAGE AND STABILITY

The test kit can be stored at 4° C to 30° C. The test strips should be kept in the foil pouch and away from heat and direct sunlight. Do not freeze.

REAGENTS AND MATERIALS SUPPLIED

Six (6) sealed protective pouches containing:

- A. Test strip
- B. Moisture absorbent packet

(This packet is for preserving the freshness of the test unit while inside the pouch. This packet is **NOT** to be used with the test and should be discarded.)

PROCEDURE

First, determine the length of your menstrual cycle. This is the number of days from the first day of menstrual bleeding to the day before the next bleeding begins again. Refer to the following chart to determine when you should start testing. The first day of bleeding is day 1.

Perform all 6 tests six days in a row or until the LH surge has been detected.

Cycle Length	Start To Test on	Cycle Length	Start To Test on
21 days	Day 6	31 days	Day 14
22 days	Day 6	32 days	Day 15
23 days	Day 7	33 days	Day 16
24 days	Day 7	34 days	Day 17
25 days	Day 8	35 days	Day 18
26 days	Day 9	36 days	Day 19
27 days	Day 10	37 days	Day 20
28 days	Day 11	38 days	Day 21
29 days	Day 12	39 days	Day 22
30 days	Day 13	40 days	Day 23

NOTE: If your cycle is shorter than 21 days or longer than 40 days, consult your doctor.


SPECIMEN COLLECTION

Choose a convenient time to collect your urine. This should be done at about the same time each day of testing. You will need 5-7 minutes to perform the test. **DO NOT USE THE FIRST URINE IN THE MORNING.** Perform the test within one hour of collecting the urine.

Collect urine sample in a dry, clean container. The container must be free of detergent.

TEST PROCEDURE:

1. Tear open sealed pouch, remove test strip. Leave the moisture absorbent packet in the pouch. (You do not need this packet for doing the test.)
2. Hold green end of test strip, dip strip into urine up to the MAX line with the arrow end pointing down. **Do not dip pass the MAX (maximum) line.**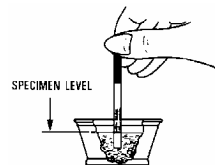

Test strip in urine sample

3. Hold for 2-3 seconds. Remove strip from urine sample and then lay it flat on a clean, dry, non-absorbent surface.
4. **Wait 5 minutes** before reading result. **Do not read results after 10 minutes.**

INTERPRETATION OF RESULTS:


While you are waiting for the your results, you may observe one band appearing first. You must wait for the full 5 minutes to see if other band will appear.

After 5 minutes, you should see a colour band appear near the upper end (where you hold the strip) of the strip. *If no band appears here, the test did not work properly.* Do not

read results, discard the test and repeat with a fresh test unit.


Positive:

If two colour bands are visible, and the lower band is **darker** than the upper band, you will probably ovulate in the next 24-48 hours.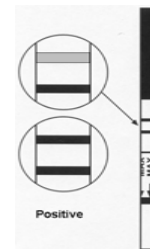

Negative:

If you see that the lower band is lighter than the upper band or only the upper band is visible, no LH surge has been detected.

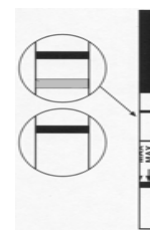EXPECTED RESULTS

A positive result indicates your urine contains a high level of LH. You can assume you will ovulate in the next 24-48 hours and plan accordingly. The 3 days before through to 3 days after ovulation is the most fertile time of your cycle when you have the best chance of conceiving.

A negative result indicates no LH surge is detected in your urine. You should continue performing the test daily.

PERFORMANCE CHARACTERISTICS

Sensitivity and Specificity

The BioStrip LH Ovulation Prediction Test has been designed to produce a definitive color band at test region when tested with 35 mIU/ml or higher of hLH (WHO 2nd IS for human luteinizing hormone) at room temperature. During the evaluation of the One Step LH ovulation test, samples containing 35 mIU/ml of hLH were tested 60 times, and definitive color bands at test region detected were more intensive than that of the control band 100% of times. Specificity of the One Step LH ovulation test was determined from cross reaction studies with known amounts of follicle stimulating hormone (hFSH), thyroid stimulating hormone (hTSH) and chorionic gonadotropin (hCG). Samples containing 200 mIU/ml hCG, 200 mIU/ml hFSH and 200 µIU/ml hTSH yielded color less intensive than that of 20 mIU/ml hLH Reference.

Accuracy

Correlation with a Qualitative Visual Test: 350 urine specimens from 70 menstrual cycles were analyzed by One Step LH test procedure in parallel with a commercial available

visual methods. The surge dates of all of these cycles, were identified and agreed by both One Step LH test and the competitor assay,

Interference Testing

The following substances were added in hLH free and hLH spiked urine samples. None of the substances at concentration tested interfered in the assay.

Acetaminophen	20 mg/dl
Acetylsalicylic Acid	20 mg/dl
Ascorbic Acid	20 mg/dl
Atropin	20 mg/dl
Caffeine	20 mg/dl
Gentamic Acid	20 mg/dl
Glucose	2 g/dl
Hemoglobin	1 mg/dl

STANDARDIZATION

The BioStrip LH Ovulation Prediction Test has been standardized to World Health Organization Second International Standard (IS 80/552).

DISPOSAL

Urine samples and used test devices are potentially infectious. Follow each laboratory's established procedure for proper handling and disposal.

REFERENCES

1. Bangham, D.R. Acta Endocrinol. 71,625-637 1972.
2. Speroff, L., Glass, R.H., Kase N.G. Clinical Gynecologic Endocrinology and infertility, 3rd ed. , Williams and Wilkins, Baltimore, MD, 1983.
3. France, J.T. In Recent Advances in Obstetrics and Gynaecology Number 14, J. Bonner, ed., Churchill Livingstone. New York, NY. 1982. pp 215-239.
4. Collins, W.P., Branch, C.M. Collins, P.O., Sallam, H.N. Int J Fert 26, 196-202 (1981).
5. Edwards, R.G., Steptoe, P.C., Purdy, J.M., Br. J Obstet Gynaecol 87, 737-756 (1980).
6. Yen, S., Vela, P., and Rankin, J., Journal of Clinical Endocrinology and Metabolism, 30,435-442 (1970).
7. Engvall, E. Method in Enzymology, Vol. 70, pp. 419-439, 1980.
8. Uotila, M., Ruoslahti, E. and Engvall, E. J. Immunol. Methods, Vol. 42, 11, 1981.

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Innovatek Medical Inc.