Innovatek Cytology Brush

INTENDED USE

The Innovatek Cytology Brush is intended for the collection of cell samples for Pap smear analysis and other cytology studies. For professional use only.

SUMMARY AND EXPLANATION

Introduced in 1941, the Pap smear is an inexpensive yet effective method of screening women for invasive, but clinical occult, carcinoma of the cervix. The systemic use of Pap smears has led to a steady reduction in cervical mortality.

The experience of the sampler, the method of sampling, and the age of the patient affect the reliability of obtaining smears. The most frequently used instruments for the Pap smear were the cervical scraper/spatulas.

Most dysplastic epithelial abnormalities are believed to arise in or near the squamocolumnar junction, and most authorities feel that smears lacking endocervical cells are inadequate for reliable interpretation. The elongated "tongue" of the spatula can be used for scraping the endocervical canal.

Some studies indicated that endocervical canal cells samples collected by using cytology brushes produce better Pap smears.

The Innovatek Cytology Brush is composed of long bristles joined to a wire shaft; the brush head on the wire shaft is bounded to a plastic handle. To collect cervical cells the brush end is inserted 1.0 cm. to 1.5 cm. into the cervical os until the longest outer bristles of the brush touch the ectocervix. The brush is rotated one full turn in a counter-clockwise direction and withdraw. The brush is then used to spread the specimen onto a glass slide for a Pap smear.

WARNING AND PRECAUTION

1. For professional use only.
2. The cytology brush is not intended for use in pregnant woman.
3. This product is for single use only. Do not reuse.
4. Contact bleeding is known to occur.
5. Do not use the brush if the bristle part has loosened from the handle.

STORAGE

Store at room temperature inside the plastic bag in a dry place. Do not store near heaters or under direct sunlight.

INSTRUCTION AND PROCEDURE :

Patients should not douche or have sexual intercourse for at least 24 hours prior to examination and should not be menstruating.

Collect sample with scraper first before collecting endocervical canal sample with cytology brush.

For Pap Smears :

1. After sampling the vaginal fornix and ectocervix with a cervical scraper/spatula, gently insert the brush into the endocervix until only the bristles closest to the handle are visible.
2. Slowly rotate one full turn.
3. Withdraw the brush.
4. Roll (do not smear) the brush with moderate pressure across a glass slide.
5. Fix with a cytology fixative immediately and discard the brush.

LIMITATIONS

Pap smear:

1. Contact bleeding in 2-3% of cases may interfere with subsequent colposcopic examination.
2. Cytology specimen must be fixed immediately with proper cytofixative to avoid air drying artifact.

(Cytology spray fixative is available from Innovatek Medical)

DISPOSAL

All materials that have been in contact with the patient are potentially infectious. Follow each clinic/laboratory's established procedure for proper handling and disposal.

REFERENCE

1. Koss LG. Diagnostic Cytology and its Histopathologic Bases, J.B. Lippincott Co.. 1979.
2. L. Szalay Wilhelm Maudrich : Cytology of the uterine cervix. Medical Publishers Wien-Miincheu-Bern. 1990.
3. Schuman J, O'Connor DM. Pap Smear Collection Devices. Diagn Cytopathol. 1992.
4. Hoffman MS, Cavanagh D. Cervical Cancer: Screening and Prevention of Invasive Disease. Cancer Control Journal. 2:6, 1998.
5. Aaen V, Ferry L. Can Cervical cancer screening be improved. Federal Practitioner. 1998.

Cat. #4201-CB-8B Cytology brush, blue handle
Cat. #4201-CB-8L Cytology brush, lavender handle

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Innovatek Medical Inc.