



# A SMARTER TEST FOR TODAY'S NEEDS

SAME TRUSTED PANBIO™ BRAND, NOW WITH EXPANDED  
DETECTION FOR COVID-19 AND FLU A/B

## PANBIO™ COVID-19/FLU A&B PANEL SELF TEST

OVER **90%**  
ACCURATE



For patients presenting symptoms  
or suspected of current **COVID-19**,  
**Flu A and/or B** infection



Uses single, patient-friendly nasal swab



For symptomatic individuals,  
aged 2 years or older\*



Results in 15 minutes




**Reduced environmental impact** with  
smaller and lighter kit boxes that  
make shipping and storage easier


**ABBOTT HAS A RICH LEGACY AS A TRUSTED BRAND IN MORE THAN 100 COUNTRIES.**

\*Individuals below 16 years of age and persons with reduced physical, sensory or mental capabilities, should be swabbed and supervised by an appropriate person responsible for their safety.


## DESIGNED FOR EASY TESTING

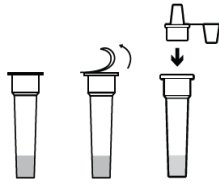

- Pre-filled buffer tube with integrated dropper cap. No manual filling needed

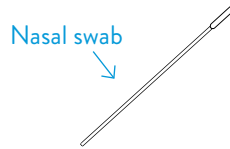


- Only one nasal swab specimen needed

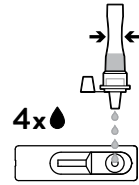

- Simple test cassette design with one sample well

## SIMPLIFIED TEST PROCEDURE

Consult Instructions for Use for complete procedure.

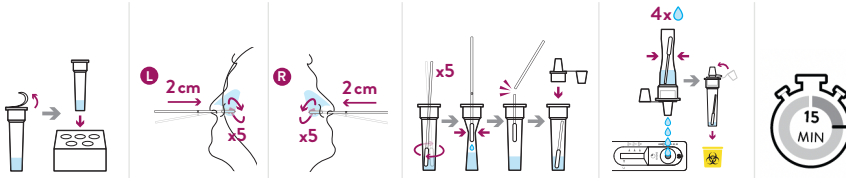

## SPECIFICATIONS

- **TEST TIME:** 15 MINUTES
- **STORAGE:** 2–30°C (shelf life 24 months)
- **SAMPLE TYPE:** Nasal swab sample

## HOW ACCURATE IS THE PANBIO COVID/FLU A&B PANEL SELF TEST AT CT VALUES ≤ 35?

**Panbio™ COVID-19/Flu A&B Panel Self Test** is an excellent lateral flow test with demonstrated sensitivity for COVID-19 of 86.3% in Ct values ≤ 35 and overall specificity of 99.2% for COVID-19. For Influenza, **Panbio™ COVID-19/Flu A&B Panel Self Test** demonstrated a sensitivity of 89.3% for Influenza A and B combined in Ct values ≤ 35, as well as an overall specificity of 98.6% for Influenza A and B combined.

## SUSTAINABILITY



### UP TO 33% SMALLER AND 20% LIGHTER BOXES REDUCE ENVIRONMENTAL IMPACT

Abbott is committed to improving the sustainability of our products by eliminating material waste. The **NEW AND IMPROVED**, smaller and lighter kit boxes make shipping and storage easier.

#### PANBIO COVID-19 AG RAPID TEST DEVICE – OLD PACKAGING

- 220x68x30mm



#### PANBIO COVID-19/FLU A&B PANEL – NEW PACKAGING

- 120x55x25mm



The included Nasal Swab is now 7cm shorter, allowing for smaller kit sizes and reduced waste.

15cm



8cm

Special “break-off” swab stays contained in tube, minimizing exposure.



**INTENDED PURPOSE:** **Panbio™ COVID-19/Flu A&B Panel Self Test** is a manual test and is intended to be used by untrained lay users (self testing) in a home setting to aid in the diagnosis of an active COVID-19, Flu A or Flu B infection.

## ORDER INFORMATION

### Panbio™ COVID-19/Flu A&B Panel Self Test

### Test format

62FK531

1 Test

62FK731

4 Tests

## CONTACT YOUR LOCAL REPRESENTATIVE TODAY.

[global.pointofcare.abbott](https://global.pointofcare.abbott)

© 2025 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. RDx-25003261-02 08/25

